



ANDERSEN
S T E R I L I Z E R S

Your Clinic's EO Sterilisation Journey

1. Specialist Instruments Requiring EO Sterilisation

Your practice has invested in surgical equipment that requires Ethylene Oxide (EO) sterilisation, including delicate and specialist instruments such as endoscopes, laparoscopic kits, fibre optics, cameras, and fine surgical instruments.



2. Speak With Our Sterilisation Team

Call Andersen Sterilisers on **+44 (0)1255 428 328** and speak with our Sterilisation Supervisor, who will guide you through the process and answer any questions you may have.



3. Clinic Setup & Account Creation

Once you are happy with the process, we will set up your clinic and explain how to prepare and send your instruments for EO sterilisation.

4. Prepare & Package Your Instruments

All instruments and surgical kits must be:

- Surgically cleaned
- Fully dried
- Packaged in Tyvek self-seal pouches, CSR wrap, or similar approved packaging

This ensures the equipment is ready for the EO sterilisation process.



5. Complete Your Load Sheet

A load sheet must be completed with each order to:

- Confirm quantities and contents
- Ensure traceability
- Cross-check order information for accuracy



6. Courier Collection Arranged

Once your instruments are packaged and ready, we can arrange collection through our DHL courier service.*

Alternatively, practices are welcome to use their own preferred courier service.



7. Fast Turnaround

Our turnaround time is typically 2 working days, including the day of receipt. Your sterilised instruments will then be safely returned to your practice.

8. Sterility Assurance Guaranteed

All sterilisation bags contain dosimeters and orders are only released once a positive result has been confirmed, ensuring guaranteed sterility assurance.



9. Aeration Before Use

Upon receiving your sterilised instruments, we recommend allowing items to aerate for 24 hours before use. This allows any residual EO gas to fully dissipate.



10. Simple Pricing

£80 per sterilisation bag

Each bag holds up to 8kg of instruments

For first orders, payment is required in advance. After this, invoices are issued on 30-day payment terms.

Additional Information

* DHL courier service is available at an additional cost of £33 each way.

* Courier collections can be arranged for UK customers only.

Visit Us at BVA Live

BVA Live 2026

 Stand B25

 11th–12th June 2026

 NEC Birmingham

Our team will be at hand to provide live demonstrations, answer any questions you may have, provide quotes and explain all the benefits of EO sterilisation.

We are also running a competition where you could be in with a chance of winning a £20, £30 or £50 Amazon voucher!

We look forward to seeing you there!



<https://www.anderseneurope.com/>



uk@anderseneurope.com



[+44 \(0\)1255 428 328](tel:+44(0)1255428328)



[Connect with Us](#)

